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USING THE VISUAL TEST KNOWN AS MONOYER 3 or 5 meters

REF 32 810 03 and 32 810 05

We thank you for choosing the eye chart made by COMED and we hope that you will be completely satisfied.

In the package you will find: a scale consisting of 3 plates, 4 pre-mounted connecting rings, 1 chain with pre-assembled attachment and the user manual.

1 / Type of installation.

Remove the scale from the packaging and hang it using the chain at a height of about 180 cm. **CAUTION :** Take care not to leave the plastic bag within reach of children; it is not a toy and can be dangerous. Unfold the ladder carefully to avoid scratching the plates. Avoid backlight from windows, the side lighting, all lighting which can alter the normal vision of the test. Also make the lighting level is at least 88 cd / m². Check that the ladder is flat against the wall (no angle).

2/ Method of Use.

The subject should stand at a distance (D) as indicated on the ladder (either 3 or 5 meters). Preliminarily test the vision in binocular mode and proceed subsequently by eye: by masking the untested eye with an opaque object. The line is considered read and interpreted correctly if more than 50% of optotypes displayed are read. The numbers transcribed to the right of the lines indicate correspondence in tenth for a distance D.

The numbers to the left indicate the distance in metres at which the line is to be read for an acuity of 10/10.

CAUTION : The use and interpretation of tests of visual acuity is only for professionals with specific training.

3/ Monoyer scale :

It is based on the capital letters of the Latin alphabet. The measured acuity is one fifth of the angle at which the letter is seen, i.e. the thickness of the black line. With this scale the subject does not try to distinguish a small detail, but rather to recognise a known shape, which is why we talk about morphoscopic acuity, the criterion is not the separable minimum, but the legible minimum. In fact, these scales are used to determine ametropia and quantify acuity with relative precision which is usually clinically sufficient.

The optotypes are read at 3 or 5 metres. They are represented by uppercase letters of increasing size, comprising from 10 to 12 acuity values and ranging from 0.1 to 1.0 or 1.2. This scale favours the measure of good visual acuity at the expense of low visual acuity. Due to its discontinuous progression: there are as many lines to represent the range between 1 angle minute and 2 angle minutes as there are between 2 and 10 angle minutes.

Typography and graphics intervene, favouring confusion between letters of similar shape, such as the V-T-Y group which presents itself as an inverted triangle, or the N-M-H, F-P-R and K-X groups. It seems that the letters CDEFHKMNORSUVXZ are nearly equally legible.

4/ This scale is not a medical device with measuring function according to Directive 93/42. It does not replace a visual diagnostic test. It indicates a pre-diagnosis of relative accuracy.

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