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We thank you for choosing the visual acuity chart manufactured by COMED and we hope that it will satisfy you.

You will find in the package:

- A chart composed of 3 plates.
- 4 connecting rings already assembled.

- 1 chain with fastener already assembled
- One instructions for use

1/ Instructions for assembly

Take the chart out of its packaging while taking care not to leave the plastic bag in the reach of children which is not a toy and can be dangerous. Fold the chart carefully so as not to scratch the plates and hang the chart on the wall by means of the chain at around a height of 180 cm. Avoid back light or bias lights or any light that can interfere with normal viewing of the test and a minimum brightness level of 88 cd/m². Check that the chart is flat on the wall, without angles.

2/ Instructions for use

Use as well as interpretation of visual acuity tests is reserved only to professionals with specific training, opticians, ophthalmologists, doctors etc...

The subject must be at a distance equal to D indicated on the chart, or 5 meters.

First test the vision in binocular mode and then proceed successively on each eye by masking the eye not tested by an opaque object.

If more than 50% of the displayed optotypes from a row are read, it is considered as read and correctly interpreted.

The numbers transposed on the right of the rows indicate the correspondence in a tenth part for a distance D. The choice of the chart must correspond to the reading ability of the subject.

3/ Armaignac chart REF 32 815 05

It is based on Raskin E and Landolt C and the measured acuity corresponds to an angle of 5 minutes of arc under which the letter is seen, the thickness of the black line corresponds to an angle of 1 minute for it. With this chart, the subject seeks to distinguish the breaking of optotype, that is why we speak of « angular » visual acuity, which allows you to determine the resolving power of the retina.

The optotypes are read at 5 meters and if they are read at this distance, the visual acuity is equal to 10/10.

4/ Chart for children REF 32 816 05

With this chart, the subject does not seek to distinguish a small detail but to recognize a known form, that is why we speak of morphoscopic acuity, the criterion is not the minimum separable but the minimum legible. In reality, these charts are used to determine ametropia and to quantify an acuity with relative accuracy which is usually enough in clinical practice. Optotypes consist of representative drawings of common objects or animals as well as geometric signs geometric.

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