

24h URINE CONTAINER 2.500ml - Technical Sheet

General description	Graduated urine container. 2500 ml capacity with carrying handle, opens with stopper for the collection and has a small hole for emptying stopper. Waterproof unsterile.		
Sterilization	Not		
Autoclavable	Not		
Purifying	Yes	In the washing machine or by hand with a cleaning product that does not attack the HDPE Polyethylene and Polypropylene PP	
Inappropriate Substances	Products that attack HDPE and PP		
Individual parts	Yes	1 bottle, 1 plastic handle, 1 large cap, 1 small cap	
Materials	HDPE polyethylene container that can withstand from 80 to 120 degrees for a short time, handle and caps from polypropylene resistant to temperatures above 120 degrees C		
Packaging	Yes	28 pieces per carton	
Marking the DM	Yes	Marking CE, recyclable, maximum occupancy 2500 ml, gradient of 100 ml and numbered each 500 ml	
Dimensions	Diameter = 132	Height = 254 With the handle = 310	Large cap's orifice = 75 Small cap's orifice = 11
Temperature resistance	Max = +80 Min = -10		
Net Weight (g)	130		
Capacity (ml)	2.500		
Colour	White		
Use	Allows the 24h urine collection		
Instructions of use	Open the large stopper for collection of urine and reclosed after use. For draining open the small cap and close it after use. Excludes instructions because of the ease of use.		
Precautions for use	Close tightly the caps to ensure an excellent seal the container. Do not fill to maximum. Container for personal use, not given to another if not done previously disinfection of all parties.		